

aminogam®

mouthwash

Adjuvant in the processes of
gingival mucosa regeneration

**DO NOT USE FOR APPLICATIONS OTHER THAN THOSE
INDICATED IN THE PATIENT INFORMATION LEAFLET**

INTENDED USE

AMINOGRAM® mouthwash is a medical device produced in compliance with Directive 93/42/EEC MDD. The product contains hyaluronic acid, which is the main component of connective tissue, and amino acids that form collagen. AMINOGRAM® mouthwash supports the tissue regeneration process and promotes rapid wound healing and simultaneous pain reduction.

DESCRIPTION

A highly biocompatible liquid that protects by promoting more rapid tissue regeneration and facilitates physiological re-epithelization of the damaged area by stimulating fibroblast growth

COMPOSITION

purified water, sodium hyaluronate, glycine, L-proline, L-leucine, L-lysine HCl, methyl parahydroxybenzoate, propyl parahydroxybenzoate, propylene glycol, mint aroma, polyvinylpyrrolidone K-90, saccharine, polysorbate 20, E131 patent blue colouring 85%, tetrasodium EDTA.

PACKAGING

200ml bottle.

FIELDS OF APPLICATION

- a) Glossitis.**
- b) Gingivitis.**
- c) Gingivitis caused by mechanical irritation from mobile prostheses.**
- d) Burning mouth syndrome.**
- e) Irritation of the oral mucosa.**
- f) Chronic inflammation.**
- g) To reduce gingival and dental sensitivity after curettage.**
- h) Protection of surgical wounds in dentistry.**

INDICATIONS FOR USE

After cleansing the oral cavity, rinse the mouth several times. It is always a good idea to avoid acidic or spicy foods. Drinking or eating or rinsing your mouth is not recommended for at least one hour after application.

CONTRAINDICATIONS

Do not use in the case of known hypersensitivity to the ingredients.

There are no known overdose events.

There are no known side effects.

There are no known interactions with drugs and medicinal substances.

Although no secondary effects are known, in the event of pregnancy and lactation, a doctor must decide whether it is advisable to prescribe the application.

AMINOGAM® mouthwash is a well-tolerated product.

WARNINGS

- The expiry date refers to the product when properly stored in a sealed package
- Do not use after the expiry date
- Do not sterilise
- Do not use if the package is open or damaged
- Do not use in the event of known hypersensitivity to the ingredients
- Keep out of reach of children
- In the event of adverse reactions, discontinue treatment and consult your doctor
- involuntary ingestion of the product will not harm to the patient

DOSAGE AND ADMINISTRATION

Apply three to four times daily, distributing the product evenly throughout the entire oral cavity and keep in mouth for at least one minute.

The product can be used pure or diluted, depending on the severity of the symptoms.

DISPOSAL

Do not release into the environment.

STORAGE

AMINOGAM® mouthwash must be stored at a temperature between 8°C and 30°C.

Do not store in a refrigerator. Store in a cool, dry place away from moisture and heat.

MANUFACTURER

PROFESSIONAL DIETETICS S.p.A. - Via Ciro Menotti, 1/A- 20129
Milan- **Italy**